

WASHER / DISINFECTOR



CS1 ST

CS2 ST

CS3 ST

CS3 HD

Pulp support

leaders in infection control equipment for 60 years
www.standbridge.co.uk


Stanbridge Limited, Beaumont House, Cray Valley Road,
St. Mary Cray, Orpington, Kent BR5 2UB UK
Tel +44 (0) 1689 806500

Stanbridge specialises in sluice room and infection control equipment.

Established in the 1950s and credited with pioneering design for modern washer/disinfector machines, Stanbridge has the expertise and practical experience that can assist and reassure in every situation.

- infection control solutions for hospitals and care homes
- involvement in development of infection control procedures
- advising on sluice room requirements
- training programmes for engineers
- advice on HTM 2030 and BSEN ISO 15883-1 and -3
- participation in seminars and study days
- support and encouragement for the care sector

The comprehensive Stanbridge range includes washer/disinfectors, stainless steel products for sluice rooms, bespoke equipment and fittings, and utensils.

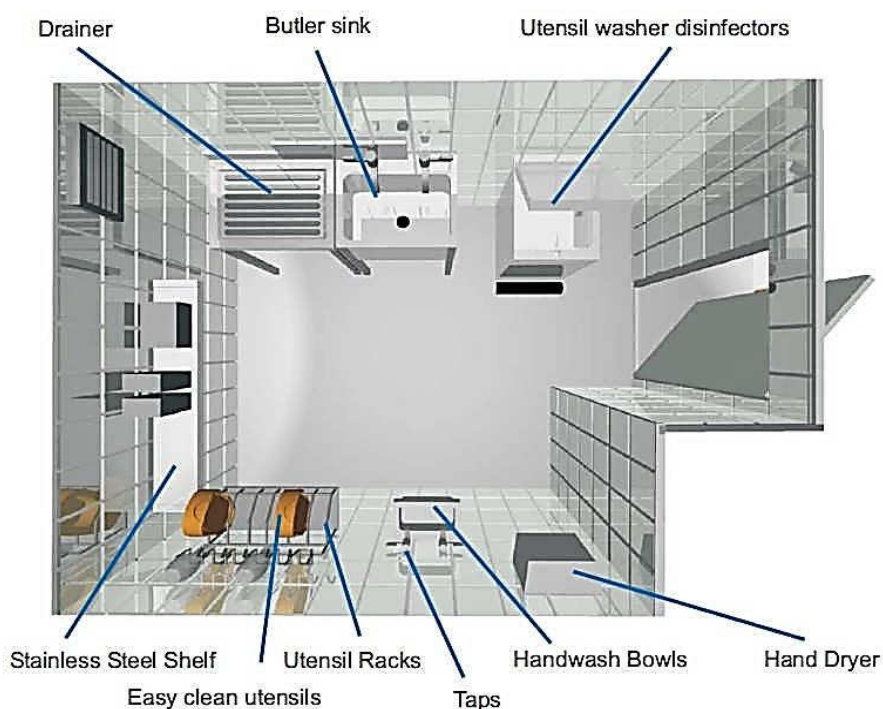

FOLLOW US ON



leaders in infection control equipment for 60 years

Stanbridge Limited, Beaumont House, Cray Valley Road,
St. Mary Cray, Orpington, Kent BR5 2UB UK
Tel +44 (0) 1689 806500

www.stanbridge.co.uk





The CS1 ST WASHER / DISINFECTOR



The Compact Superior 1 washes and disinfects utensils one at a time and is specially designed for the small nursing home where space is limited.

The CS1 provides an effective infection control barrier which protects residents and staff from the risk of cross infection and management from potential liabilities. All Stanbridge washer/disinfectors can include the DEOSCALE water treatment system to control limescale build-up and reduce maintenance

Stanbridge utensil washer disinfectors are manufactured in compliance with the recommendations of British Standard 2745 and to meet the detail requirements of inspectors of the Water Supply Undertakings

Stanbridge Ltd is approved under BS 5750 for quality assurance.

WRAS Approval No. 0310064 Internal break tank has a type 'A' air gap supply



Size
Height
Width
Depth

Installation
910mm (36")
435mm (18")
660mm (26")

Electrical

230V 50Hz 16A single phase
Fuse - mains 16A - control 5A

Water Feed
Supply
Connection

Cold
1/2" BSP male parallel

Washing
cycles

1. Normal wash
Wash - Disinfection at
80°C for 70 seconds -
Rinse to cool

Overflow

1x19mm (3/4") bore.
End must be visible.
(UK requirement)

2. Super wash
Pre flush - Wash -
Disinfection at 80°C for 70
seconds - Rinse to cool

Waste

82mm (3") 'P' or 'S' trap



The CS2 ST WASHER / DISINFECTOR



The Compact Superior 2 is specifically designed for multi-storey care homes where code of practice requires a sluice room on each floor

The CS2 medium sized top loading machine washes and disinfects a variety of utensils two at a time. For care homes committed to disposable products it is an ideal, versatile washer/disinfector for the carriers and reusable utensils to complete the infection control barrier. All Stanbridge washer/disinfectors can include the DEOSCALE water treatment system to control limescale build-up and reduce maintenance.

Stanbridge utensil washer disinfectors are manufactured in compliance with the recommendations of British Standard 2745 and to meet the detail requirements of inspectors of the Water Supply Undertakings.

Stanbridge Ltd is approved under BS 5750 for quality assurance.

WRAS Approval No. 0310064 Internal break tank has a type 'A' air gap supply



Size
Height
Width
Depth

Installation
910mm (36")
435mm (18")
660mm (26")

Electrical

230V 50Hz 16A single phase
Fuse - mains 16A - control 5A

Water Feed
Supply
Connection

Cold
1/2" BSP male parallel

Washing
cycles

1. Normal wash
Wash - Disinfection at
80°C for 70 seconds -
Rinse to cool

Overflow

1x19mm (3/4") bore.
End must be visible.
(UK requirement)

2. Super wash
Pre flush - Wash -
Disinfection at 80°C for 70
seconds - Rinse to cool

Waste

82mm (3") 'P' or 'S' trap



The CS3 ST WASHER / DISINFECTOR



The Compact Superior 3 ST Commode Pot & Bottle machine is designed for the full bed occupancy and high dependency nursing homes which have peak loads and a high utensil volume.

The CS3 ST has the capacity to efficiently wash and disinfect 3 utensils in one operation, cutting water and heating costs and staff waiting time and utensil queueing. All Stanbridgewasher/disinfectors can include the DEOSCALE water treatment system to control limescale build-up and reduce maintenance.

Stanbridge utensil washer disinfectors are manufactured in compliance with the recommendations of British Standard 2745 and to meet the detail requirements of inspectors of the Water Supply Undertakings

Stanbridge Ltd is approved under BS 5750 for quality assurance.

WRAS Approval No. 0310064 Internal break tank has a type 'A' air gap supply



Size	Installation	Electrical	230V 50Hz 16A single phase
Height	975mm (39")		Fuse - mains 16A - control 5A
Width	600mm (24")		
Depth	710mm (28")		
Water Feed Supply Connection	Cold 1/2" BSP male parallel	Washing cycles	1. Normal wash Wash - Disinfection at 80°C for 70 seconds - Rinse to cool
Overflow	1x19mm (3/4") bore. End must be visible. (UK requirement)		2. Super Wash Pre flush - Wash - Disinfection at 80°C for 70 seconds - Rinse to cool
Waste	82mm (3") 'P' or 'S' trap		



The CS3 HD WASHER / DISINFECTOR





The Compact Superior 3 HD **multi-utensil** machine is designed for the full bed occupancy and high dependency nursing homes which have peak loads and a high utensil volume.

The CS3 HD has the capacity to efficiently wash and disinfect 3 utensils in one operation, cutting water and heating costs, staff waiting time and utensil queueing.

All Stanbridge washer/disinfectors can include the DEOSCALE water treatment system to control limescale build-up and reduce maintenance. Stanbridge utensil washer disinfectors are manufactured in compliance with the recommendations of British Standard 2745 and to meet the detail requirements of inspectors of the Water Supply Undertakings

Stanbridge Ltd is approved under BS 5750 for quality assurance.

WRAS Approval No. 0310064 Internal break tank has a type 'A' air gap supply

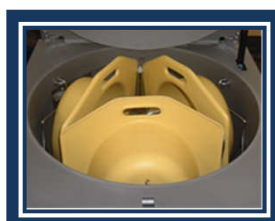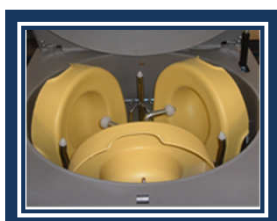

Size
Height
Width
Depth

Installation
1100mm (44")
600mm (24")
710mm (28")

Electrical

230V 50Hz 16A single phase
Fuse - mains 16A - control 5A

Water Feed
Supply
Connection

Cold Feed x 2
1/2" BSP male parallel

Washing
cycles

1. Normal wash
Wash - Disinfection at 80°C for 70 seconds - Rinse to cool

Overflow

1x19mm (3/4") bore.
End must be visible.
(UK requirement)

2. Super Wash
Pre flush - Wash - Disinfection
at 80°C for 70 seconds -
Rinse to cool

Waste

82mm (3") 'P' or 'S' trap



Stanbridge Limited, Beaumont House, Cray Valley Road,
St. Mary Cray, Orpington, Kent BR5 2UB UK
Tel +44 (0) 1689 806500 www.stanbridge.co.uk





PULP SUPPORT WASHER/DISINFECTOR



For ultimate infection control the rigid support carrier for cardboard liner bed pans must be treated as a reusable utensil.

The Stanbridge range of washer/disinfectors are perfect to ensure reusable carriers are washed and disinfected to the standards detailed in HTM 20/30 and BS EN ISO 15883.

For your peace of mind the Stanbridge range of washer/disinfectors provide the solution to all user needs for reusable products, bedpans, commode pots, bowls, bottles, jugs and support carriers ensuring clean and disinfected utensils every time.





CELEBRATING 60 YEARS

Stanbridge Limited,
Beaumont House, Cray Valley Road,
St. Mary Cray, Orpington, Kent BR5 2UB UK

+44 (0) 1689 806500

leaders in infection control equipment for 60 years

